ODSTRANJEVALEC ALGINATA

natrijev hidroksid

SESTAVA

Prah vsebuje natrijev hidroksid, natrijev karbonat, dikloroizocianurno kislino-natrijevo sol.

DELOVANJE

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natrium hydroxide

COMPOSITION

The powder contains natrium hydroxide, natrium carbonat, Dichloroisocyan acid- Na salt.

ACTION

Alginat eliminator solution removes all alginat and other organic and anorganic residues from the impression trays and metal, plastic, glass and porcelan instruments. It is not used for cleaning of aluminium instruments.

DOSAGE AND APPLICATION

Six teaspoonfuls (50 g) of the powder is dissolved in 1 litre of water and thoroughly mixed. The impression trays and instruments should be kept in the solution at least for two hours, thoroughly washed with tap water and dried. The solution can also be used in an ultrasound chamber for cleaning of the instruments.

WARNING

When using the solution care should be taken to protect the hands and to avoid contact with the eyes.

The solution is strong alkalik. It is kept in tight sealed plastic containers in a dry place.

FORM OF SUPPLY

Plastic containers of 700 g powder.

DANGER

Causes severe skin burns and eye damage. / May cause respiratory irritation. / Very toxic to aquatic life with long lasting effects. / Do not breathe dust/vapours. / Wash hands thoroughly after handling. / Avoid release to the environment. / Wear protective gloves/protective clothing/eye protection/face protection. / IF SWALLOWED: rinse mouth. Do NOT induce vomiting. / IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. / IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. / Dispose of contents/ containers in accordance with local regulation.

SI - 3000 CELJE · Opekarniška cesta 26 · Tel.: +386 (0)3 425-62-00

Fax: +386 (0)3 425-62-10 · E-mail:info@interdent.cc · http://www.interdent.cc