# TERMO PLOŠČA / THERMO PLATE WARMEPLATTE / TERMO PLOČA ТЕРМОАГРЕВАТЕЛЬ

## REF 0616

 Navodila za uporabo /  Directions for use  
 Upute za upotrebu /  Gebrauchsinformationen  
 Инструкции по применению

**CZ**  
INTERDENT s.r.o.  
Foerstrova 12, Strašnice  
CZ-10000 Praha  
T: +420/274 783 114  
F: +420/274 820 130  
E: interdent@interdent.cz

**HR**  
INTERDENT d.o.o.  
Vinogradski odvojak 2d  
HR-10431 Sveta Nedelja  
T: +385/1 3873 644  
F: +385/1 38736 17  
E: interdent@interdent.hr

**SRB**  
INTERDENT d.o.o.  
Zemunska 22, lok 3  
RS-11070 Novi Beograd  
T/F: +381/11 217 53 74

**MK**  
INTERDENT d.o.o.  
Bojmija 1-12M  
MK-1000 Skopje  
T: +389/224 628 55  
F: +389/224 634 12  
E: intsk@t-home.mk

**SK**  
INTERDENT SK, s.r.o.  
Liščie údolie 57  
SK-84231 Bratislava  
T: 00421 2 5440554,  
F: 00421 2 5440555  
E: interdent@interdent.sk

**INTERDENT®**

Proizvajalec/Producer/Hersteller  
SI - 3000 CELJE · Opekarniška cesta 26  
T: +386 (0)3 425-62-00 · F: +386 (0)3 425-62-10  
E: info@interdent.cc · www.interdent.cc

Verzija: 02/2015

Made in Slovenia

## TERMO PLOŠČA

### REF 0616

#### SI

#### NAVODILO ZA UPORABO

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#### DIRECTIONS FOR USE

##### 1. Usage area

Thermo plate is suitable for use in dental offices for heating wax for bite registration and also in dental laboratories for heating wax sheets, pink wax and biting edges.

##### 2. Advantages

Easy to use, precise mechanical temperature control.

##### 3. Technical data

Connection:	230 V, 50/60 Hz
Power take away during heating:	85 W
Power taken away during work:	12 W
Temperature range:	30 - 60°C without level regulation
Function signalization:	lightened switcher
Outside measurements:	width=160 mm, depth=80 mm, height=70 mm
Weight	0,7 kg
Cable with protective connector:	1,5 m

##### 4. Description

Thermo plate is a compact, efficient heating plate, which is made of PTFE material. Mechanically operating working temperature with an accuracy of  $\pm 0,5^\circ\text{C}$ , allows the correct preheat of the wax and thus prevents undesired internal wax tension. Temperature control covers the range from 30 to 60°C.

##### 4.1. Before first use

When you take new device from the box make sure it is not damaged from the transport. If you notice any damage return the device immediately. Put the device on desired place and connected it with the plug on heating container. Make sure that the net voltage is in accordance with data on the board of device. If it is appropriate than connect the device on the net.

##### 4.2. Usage

##### 4.2.1. Instruction

Turn on the device with switcher. Green light on the switcher show you that device is ready. Desired temperature is placed with swivel knob. After this thermo plate can be used.

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##### 4.2.2. Mistakes removing

If thermo plate is cold even when the green light is on, must mistake be removed by authorized service.

##### 4.3. Maintenance

The device should be clean. Repairing of it must be done by authorized service. Interdent is not responsible for mistakes made by unprofessional. Guarantee for device is one year and does not valid in case of using the device in other purposes that are described in this directions for use and in case of intervention in device by unauthorized service.

##### 5. Safety precaution

At direct contact with the hot plate burns can occur. Do not place any object on the plate. You can mechanically damage the PTFE plate with sharp object. Interventions in device can be made only by professionals. Do not use device on people.

##### 6. Content

REF 0616 Thermo plate 1 pcs

##### 7. Accessories

/

##### 8. Warranty

Our recommendations and instructions are based on our own experiences. Using of device in any other purposes as described above is made on own responsibility. Quality of device is tested and assured.

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The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided

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Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired.

- Veljavnost garancije dokazujete s potrjenim garancijskim listom in original računom.

Validity of the warranty should be proofed by certified warranty card and original invoice

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