TERMOTOP / THERMOTOP ТЕРМОТОП REF 0601

 Navodila za uporabo /  Directions for use
 Upute za upotrebu /  Gebrauchsinformationen
 Инструкции по применению

CZ
INTERDENT s.r.o.
Foerstrova 12, Strašnice
CZ-10000 Praha
T: +420/274 783 114
F: +420/274 820 130
E: interdent@interdent.cz

HR
INTERDENT d.o.o.
Vinogradski odvojak 2d
HR-10431 Sveta Nedelja
T: +385/1 3873 644
F: +385/1 38736 17
E: interdent@interdent.hr

SRB
INTERDENT d.o.o.
Zemunska 22, lok 3
RS-11070 Novi Beograd
T/F: +381/11 217 53 74

MK
INTERDENT d.o.o.
Bojmija 1-12M
MK-1000 Skopje
T: +389/224 628 55
F: +389/224 634 12
E: intsk@t-home.mk

SK
INTERDENT SK, s.r.o.
Lišcie údolie 57
SK-84231 Bratislava
T: 00421 2 5440554,
F: 00421 2 5440555
E: interdent@interdent.sk

INTERDENT®

Proizvajalec/Producer/Hersteller
SI - 3000 CELJE · Opekarniška cesta 26
T: +386 (0)3 425-62-00 · F: +386 (0)3 425-62-10
E: info@interdent.cc · www.interdent.cc

Verzija: 02/2015

Made in Slovenia

TERMOTOP

REF 0601

SI

Aparat za potopno metodo

NAVODILO ZA UPORABO

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Termotop je električni aparat, z individualno nastavitvijo temperature grejta posodice napolnjene s potopnim voskom za izdelavo kapice v zobnem laboratoriju. S primernim voskom dobimo izredno dobre in precizne rezultate.

2. Prednosti

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DIRECTIONS FOR USE

1. Usage area

Thermotop is an electrical device with individually adjustable heating temperature of the container filled with dipping wax for cap making method in the dental laboratory. With suitable wax we achieve extremely good and precise results.

2. Advantages

Simple use, efficient, with electronically regulated temperature, always prepared to be used.

3. Technical data

Connection	230V, 50/60 Hz
Power taken away at max. temperature:	heating phase 85W, working 12 W
Temperature range:	60 - 105°C without level regulation
Function signalization:	lightened switcher
Outside measurements:	width=82 mm, depth=80 mm, height=95 mm
Weight:	0,5 kg
Cable with protective connector:	1,5 m

4. Description

Thermotop is a compact, powerful device for making the caps with a dipping method. Individual temperature control of the container allows accurate result with dipping method. Temperature regulation comprises a range between 60-105°C.

4.1. Before first use

When you take new device from the box make sure it is not damaged from the transport. If you notice any damage return the device immediately. Put the device on desired place and connected it with the plug on heating container. Make sure that the net voltage is in accordance with data on the board of device. If it is appropriate than connect the device on the net.

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4.2. Instruction

4.2.1. Direction

Fill the pot with suitable wax. Switch the device on. Green light on the switcher shows that the device is ready. Regulate the desired temperature with swivel knob. Be careful! Touching the edge of the pot can cause burns.

4.2.2. Mistakes removing

If the pot overheats, call authorized service, that will find and remove the mistake.

4.3. Maintenance

Cleaning of the pot is very important Maintain clean device. Repairing of device must always perform authorized service. Interdent is not responsible for mistakes made by unauthorized services. Guarantee for device is one year and does not valid in case of using the device in other purposes that are described in this directions for use and in case of intervention in device by unauthorized service.

5. Safety precaution

Danger of burns at direct contact with pot for wax melting. The intervention in device can be made by qualified person. Do not use device on people.

6. Content of supply

REF 0601 Thermotop pot for dipping 1 pcs

7. Accessories

/

8. Warranty

Our recommendations and instructions are based on our own experiences. Using of device in any other purposes as described above is made on own responsibility. Quality of device is tested and assured.

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The product will work flawless during the warranty period if being administered in accordance with its purpose and the instructions provided

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Warranty period expires after 12 months from purchase day. At your request we will repair failure or defects on the product, if you will report it within the warranty period. Failure will be resolved free of charge within 45 days from the date of failure report. The product, which will not be repaired within this deadline, will be, at your request, replaced with a new one. We will extend the warranty period for the time the product was being repaired.

- Veljavnost garancije dokazujete s potrjenim garancijskim listom in original računom.

Validity of the warranty should be proofed by certified warranty card and original invoice

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