# Acrylock

CE 0197

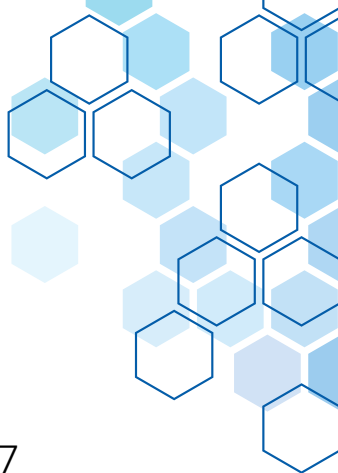
Date: 10. 6. 2026

Version: 01/2026

INTERDENT d.o.o. · Opekarniška cesta 26 · 3000 Celje · Slovenija

T: +386 3 425 62 00 · E: info@interdent.cc · www.interdent.cc

Potrebno upoštevati navodila za uporabo/ Consult instructions for use/ Bitte beachten Sie die Bedienungsanleitung/ Potrebno poštivati uputstva za upotrebu/ Viz návod k použití/ Необходимо учитывать инструкции по применению/ Συμβουλευτείτε τις οδηγίες χρήσης/ Instructions de procédure/ Consultare le istruzioni per l'uso/ Laikytis naudojimo instrukcijas/ Ievērot lietošanas pamācību/ Consultati instructiunile de folosire/ Dodržujte návod na použití/ Poštovati uputstvo za upotrebu

Uporabno do/ Use by/ Gültig bis/ Upotrijebiti do/ Použítí/ Использовать до/ Дейін қолданыңыз/ Χρήση με/ À utiliser avant/ Utilizzare entro la data/ Tinka naudoti iki/ Deriņš līdz/ Folsit de catre/ Použite do/ Upotrebljivo do

**LOT**

Kontrolna številka/ Batch code/ Seriennummer/ Serijski broj/ Seriové číslo/ Серийный номер/ Сериялык нөмір/ Κωδικός παρτίδας/ Numéro de lot/ Codice del lotto/ Partijos numeris/ Partijas numurs/ Codul lotului/ Číslo šarže/ Broj šarže

**REF**

Kataloška številka/ Catalogue number/ Katalognummer/ Kataloški broj/ Katalogové číslo/ Каталогный номер/ Каталогтық нөмір/ Αριθμός καταλόγου/ Numéro du catalogue/ Numero di catalogo/ Artikulo numeris/ Artikula numurs/ Numarul de catalog/ Číslo výrobku/ Broj artikla

**MD**

Medicinski pripomoček/ Medical device/ Medizinprodukt/ Medicinski uređaj/ Zdravotnické zařízení/ Медицинский прибор/ Медициналық құрылғы/ Ιατρική συσκευή/ Dispositif médical/ Dispositivo medico/ Medicinos priemonė/ Mediciniska ierīce/ Dispozitiv medical/ Zdravotnícka pomůcka/ Medicinski proizvod

**UDI**

Edinstveni identifikator pripomočka/ Unique device identifier/ Eindeutige Produktkennung/ Jedinstveni identifikator uređaja/ Jediněčný identifikátor zařízení/ Уникальный идентификатор устройства/ Бірегей өнім идентификаторы/ Μοναδικό αναγνωριστικό συσκευής/ Identifiant unique du dispositif/ Identificatore univoco del dispositivo/ Unikalus prietaiso identifikatorius/ Unikāls ierīces identifikators/ Identificator unic de dispozitiv/ Unikátný identifikátor pomůcky/ Jedinstveni identifikator sredstva

Proizvajalec/ Manufacturer/ Hersteller/ Proizvođač/ Výrobce/ Производители/ Өндіруші/ Κατασκευαστής/ Fabricant/ Fabbicante/ Gamintojas/ Ražotājs/ Producator/ Výrobca/ Proizvođač

Datum proizvodnje/ Date of manufacture/ Herstellungsdatum/ Datum proizvodnje/ Datum výroby/ Дата изготовления/ Өндірілген күні/ Ημερομηνία κατασκευής/ Date de fabrication/ Data di fabbricazione/ Pagaminimo data/ Izgatavošanas datums/ Data fabricatiei/ Dátum výroby/ Datum proizvodnje

Hranite na suhem/ Keep dry/ Trocken halten/ Čuvati na suhom/ Uchovávejte v suchu/ Держать сухим/ Құрғақ ұстау/ Φυλάσσετε σε ξηρό περιβάλλον/ Tenir au sec/ Tenere asciutto/ Laikyti atokiau nuo saulės šviesos/ Glabāt sausumā/ Mēģinēti uscat/ Uchovávejte v suchu/ Čuvati na suhom

Ni za ponovno uporabo/ Do not re-use/ Nicht zur Wiederverwendung/ Nemojte ponovo upotrebljavati/ Nepoužívat opakovaně/ Не использовать повторно/ Қайта пайдаланбаңыз/ Μην επαναχρησιμοποιείτε/ Ne pas réutiliser/ Non riutilizzare/ Nenaudoti pakartotinai/ Nelietot atkārtoti/ A nu se reutiliza/ Nepoužívejte opakovane/ Ne koristiti ponovo

◆ <b>SL</b>	Navodila za uporabo.....	3–4
◆ <b>EN</b>	Instructions for use.....	5–6
◆ <b>DE</b>	Gebrauchsanweisung.....	7–8
◆ <b>HR</b>	Upute za upotrebu.....	9–10
◆ <b>CS</b>	Návod k použití.....	11–12
◆ <b>RU</b>	Инструкция по применению.....	13–14
◆ <b>KK</b>	Пайдалану нұсқаулығы.....	15–16
◆ <b>EL</b>	Οδηγίες χρήσης.....	17–18
◆ <b>FR</b>	Instructions d'utilisation.....	19–20
◆ <b>IT</b>	Istruzioni per l'uso.....	21–22
◆ <b>LT</b>	Naudojimo instrukcija.....	23–24
◆ <b>LV</b>	Lietošanas instrukcija.....	25–26
◆ <b>RO</b>	Instrucțiuni de utilizare.....	27–28
◆ <b>SK</b>	Návod na použitie.....	29–30
◆ <b>SR</b>	Uputstvo za upotrebu.....	31–32

**Navodila za uporabo polzila Acrylock**

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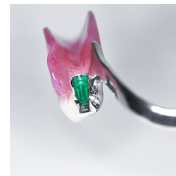
**Slika 16:** Končano delo z vstavljenim zeleno matrico.

**Instructions for use of Acrylock attachment parts**

Prior to use, carefully read the instructions for use. For use only by a dental professional within the recommended indications.

**Product description:**

Acrylock attachment parts are extracoronal, semi-precision mechanical construction elements, made of POM (polyoxymethylene), for incorporation into prostheses. They serve for rigid fix and stabilization of removable partial dentures in patients with no or little atrophied alveolar ridge. They contribute to the functional restoration of the craniomandibular chewing system. The pull-off forces can be adjusted.

**Intended use:**

Removable partial dental prostheses.

**Indications:**

Removable partial dental prostheses in the upper or lower jaw.

**Contraindications:**

- In case of known allergies to material content.
- Inability or unwillingness of the patient to follow the aftercare instructions.
- Extreme bruxism or further uncontrolled para-functional habits.

**Intended users:**

The product is intended to be used by professionals (dental technicians and dentists).

**Patient selection criteria:**

Partially edentulous adult patients.

**Precautions:**

- Acrylock attachments may only be used by qualified dentists and dental technicians with the appropriate knowledge and experience.
- The instructions must be followed.
- The components of the Acrylock attachment parts cannot be combined in conjunction with retaining elements from other manufacturers. Only original Acrylock tools and auxiliary parts must be used.
- In the event of an allergic reaction, discontinue use and consult a dentist.
- After incorporating the prosthetic restoration, regular control examinations are mandatory.
- The components are delivered in a non-sterile state. The dentures must be thoroughly cleaned before they are placed in the patient's mouth.
- To ensure a secure fit of the prosthesis and to avoid overloading of the attachment parts, the denture bearing must be checked at the six-monthly recall and the prosthesis must be relined if necessary.

- A prosthesis with loose, damaged or broken attachments should not be worn (risk of choking, swallowing, aspiration or other injuries).
- The patient must be instructed on the correct insertion and removal of the prosthesis as well as daily care of the prosthesis.

**Warning:**

If retention between the matrix and matrix is lost, the green matrix is replaced with the yellow one and at the end, the yellow matrix with the red one. Matrix replacement can be done in a dental practice or in dental laboratory.

**Instructions for cleaning:**

- All medical devices must be cleaned before use on the patient.
- The hygiene regulations of the individual national and international legal provisions of dental and medical practices, hospitals and dental laboratories must be observed.
- Do not use acidic solutions for cleaning. The product can be cleaned with a vapour cleaner.
- Concentration and cleaning time are always stated in the instructions for use of the cleaning agent.
- Use an aldehyde- or alcohol-free cleaning agent that is compatible with the material and listed on a certified list of cleaning agents.

**Side effects:**

Consider allergic reactions to the material.

**Storage:**

In a dry place, in the closed container package.

**Notice:**

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.


**Warranty:**

Whether given verbally, in writing, or by practical instructions, our recommendations for use are based upon our own experience and trials, and can only be considered as reference values. Our products are subject to further development. Therefore, alterations in the structure and composition are reserved.


## Directions for use:


**Photo 1:** Assemble the matrix and stress breaker and seal the connection with wax.

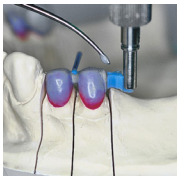**Photo 2:** Hold the matrix with the parallel holder (REF 750) and attach it to the wax-up with wax. Shorten the basal part if necessary.

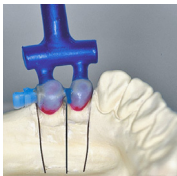
**Photo 3:** Set up the sprues in accordance with the alloy manufacturer's instructions. The yield strength of the Rp 0.2 alloy must be at least 500 N/mm<sup>2</sup>.

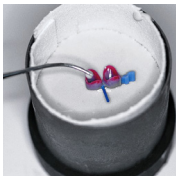**Photo 4:** Use an investment material and invest as usual.

**Photo 5:** Only polish the matrix, do not mill or cut it.

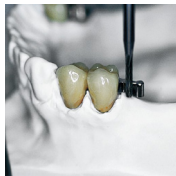**Photo 6:** Process the stress breaker with a  $\varnothing$  1.0–1.5 mm parallel cutter.


**Photo 7:** Always use a green matrix for duplication. Fill the gap under the matrix with wax.


**Photo 8:** Fill the occlusal and lateral gaps between the matrix and the matrix with wax.


**Photo 9:** Pour out the duplicated model from the investment material.

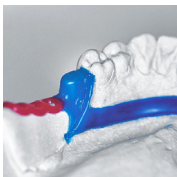
**Photo 10:** Wax up the partial denture. Cover the attachment elements with at least 0.7 mm of wax. Be careful not to damage the retention groove on the matrix!

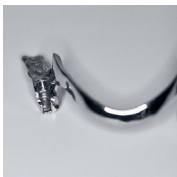**Photo 11:** Process the cast partial denture framework as usual.

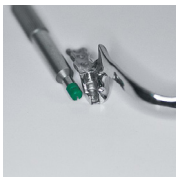**Photo 12:** Push-in pin with green matrix.

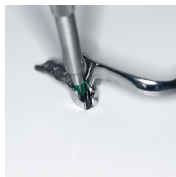
**Photo 13:** Press the matrix into the seat until it clicks into place. Attention! Note the position of the retention groove!

**Photo 14:** Polish the matrix if it is necessary to reduce friction.

**Photo 15:** Placing the teeth in wax and finishing as usual.

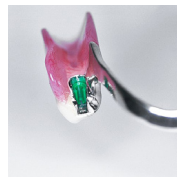**Photo 16:** Finished work with inserted green matrix.

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